

HOW TO SANITIZE YOUR SUNGLASSES AND OPTICAL FRAMES

Dear Valued Customers and Marcolin Community,

The below referenced information is being provided in an effort to assist our employees, sales representatives and customers with cleaning and handling of our eyewear frames.

HOW TO SANITIZE YOUR SUNGLASSES AND OPTICAL FRAMES

Supplies Needed

Disposable latex or vinyl gloves
Hydrogen peroxide wipes or solution
Microfiber cleaning cloth

Method

Frames can be cleaned with hydrogen peroxide wipes.

If hydrogen peroxide wipes are not available a 50-50 3% hydrogen peroxide and water solution mixed in a spray bottle can be used to clean the frames with a cleaning cloth.

Hydrogen peroxide wipes and solutions contain different ingredients from the regular disinfectant wipes and solutions. Do not use regular disinfectant wipes or solutions as these can damage the frames.

Process

1. Clean and sanitize all frames after each use.
2. Recommended to wear latex or vinyl gloves when touching the product and before starting the process.
3. Use your gloved hands to clean every part of the frames with a hydrogen peroxide wipe or spray a hydrogen peroxide solution in a regular cleaning cloth.
4. After cleaning the frames or interaction with patients, dispose the gloves and wipes. Do not reuse the cleaning cloth before machine washing it first.
5. Do not touch your mouth, nose or eyes during the process. Wash your hands for 20 seconds or use an hand sanitizer after disposing the gloves.

If you are unable to use hydrogen peroxide wipes or solution, we recommend cleaning the frames with anti-bacterial, lotion free dish-soap in a stream of lukewarm water.

Please continue to take care of yourself and each other.

Stay safe, Marcolin U.S.A.

DISCLAIMER - The above information is provided for reference purposes only and does not constitute medical advice. If you have medical related concerns about the spreading and/or contracting of infectious diseases associated with wearing or disposing of eyewear frames, please consult the Centers for Disease Control and Prevention at www.cdc.gov for the most recent developments and advice. All information is provided based on the Marcolin USA Eyewear Corp. understanding and belief as of the date on which this information was prepared and is subject to change or modification at any time in Marcolin USA Eyewear Corp.'s sole discretion. Marcolin USA Eyewear Corp. assumes no obligation or liability for the information contained herein.

HOW TO SANITIZE YOUR SUNGLASSES AND OPTICAL FRAMES

CLEAN AND SANITIZE ALL FRAMES AFTER EACH USE



1. Wash your hands for 20 seconds or use a hand sanitizer.



2. Apply latex or vinyl gloves.



3. Clean every part of the frame/s with a hydrogen peroxide wipe or spray a hydrogen peroxide solution in a regular cleaning cloth.



4. Do not touch your mouth, nose or eyes during the process.



5. After cleaning the frames or interaction with patients, dispose the gloves and wipes. Do not reuse the cleaning cloth before machine washing it first.



6. Wash your hands for 20 seconds or use a hand sanitizer after disposing the gloves.

DISCLAIMER - The above information is provided for reference purposes only and does not constitute medical advice. If you have medical related concerns about the spreading and/or contracting of infectious diseases associated with wearing or disposing of eyewear frames, please consult the Centers for Disease Control and Prevention at www.cdc.gov for the most recent developments and advice. All information is provided based on the Marcolin USA Eyewear Corp. understanding and belief as of the date on which this information was prepared and is subject to change or modification at any time in Marcolin USA Eyewear Corp.'s sole discretion. Marcolin USA Eyewear Corp. assumes no obligation or liability for the information contained herein.