

Dear Valued Customers & Eyewear Community,

The below referenced information is being provided in an effort to assist our employees, sales representatives & customers with the cleaning & handling of WestGroupe frames.

HOW TO SANITIZE YOUR EYEWEAR

Recommended Methods

① Soap & Water

We recommend cleaning all frames with anti-bacterial, lotion-free soap or dishwashing liquid in lukewarm water.

② Hydrogen Peroxide Wipes or 3% Hydrogen Peroxide Solution

If hydrogen peroxide wipes are not available, a 50-50 mix of 3% hydrogen peroxide & water solution in a spray bottle can be used to clean the frames with a cleaning cloth.

Hydrogen peroxide wipes & solutions contain different ingredients from the regular disinfectant wipes & solutions. DO NOT use regular disinfectant wipes or solutions, as these will damage the frames.

③ UVC Chamber/Box: 200nm - 280nm of ultraviolet rays

Follow the recommended guidelines of your UVC chamber/box.

Process for Cleaning with Hydrogen Peroxide

- Do not touch your mouth, nose or eyes during the cleaning process.
- Recommended to wear latex or vinyl gloves before starting the process & when touching the product.
- Use your gloved hands to clean every part of the frame with a hydrogen peroxide wipe or spray the 3% hydrogen peroxide solution onto a regular cleaning cloth.
- After cleaning the frames, dispose of the gloves & wipes. Do not reuse the cleaning cloth before machine washing it first.
- Wash your hands for 20 seconds or use an hand sanitizer after disposing the gloves.
- Clean & sanitize all frames after each use.

**Please continue to take care of yourself and each other.
Stay safe.**

For more information please visit The Vision Council at the link below


<https://thevisioncouncil.org/members/reopening-guidance>

WestGroup

CLEAN & SANITIZE ALL FRAMES AFTER EACH USE



Wash your hands for 20 secs

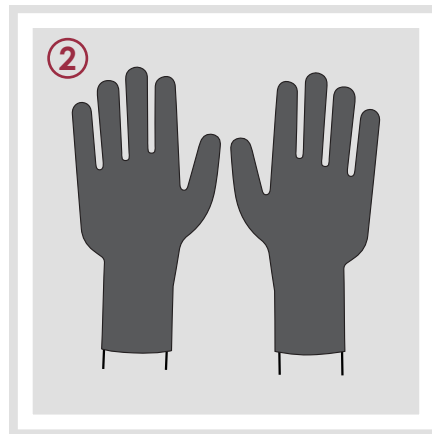


Apply latex or vinyl gloves



Clean every part of the frame(s)



DO NOT touch your face during the process

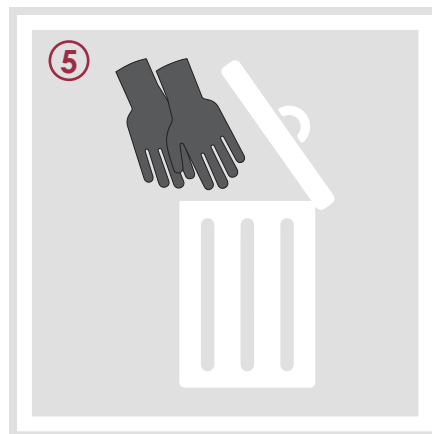

After cleaning the frames, dispose of the gloves & wipes. DO NOT reuse the cleaning cloth before machine washing it first.



Wash your hands for 20 secs after disposing of the gloves