

## HOW TO SANITIZE YOUR SUNGLASSES AND OPTICAL FRAMES

Dear Valued Customers and L'Amey America Community,

The below referenced information is being provided in an effort to assist our employees, sales representatives and customers with cleaning and handling of our eyewear frames.



### SUPPLIES NEEDED

Disposable rubber or vinyl gloves  
Hydrogen peroxide wipes or solution  
Microfiber cleaning cloth

### METHOD

Frames can be cleaned with [hydrogen peroxide wipes](#).

If hydrogen peroxide wipes are not available a [50-50 3% hydrogen peroxide and water solution](#) mixed in a spray bottle can be used to clean the frames with a cleaning cloth.

Hydrogen peroxide wipes and solutions contain different ingredients from the regular disinfectant wipes and solutions. [Do not use regular disinfectant wipes or solutions](#) as these can damage the frames.

### PROCESS

1. Clean and sanitize all frames after each use.
2. Recommended to wear rubber or vinyl gloves when touching the product and before starting the process.
3. Use your gloved hands to clean every part of the frames with a hydrogen peroxide wipe or spray a hydrogen peroxide solution in a regular cleaning cloth.
4. After cleaning the frames or interaction with patients, dispose the gloves and wipes. Do not reuse the cleaning cloth before machine washing it first.
5. Do not touch your mouth, nose or eyes during the process. Wash your hands for 20 seconds or use an hand sanitizer after disposing the gloves.

If you are unable to use hydrogen peroxide wipes or solution, we recommend cleaning the frames with anti-bacterial, lotion free dish-soap in a stream of lukewarm water.

Please continue to take care of yourself and each other.

Stay safe, L'Amey America

DISCLAIMER - The above information is provided for reference purposes only and does not constitute medical advice. If you have medical related concerns about the spreading and/or contracting of infectious diseases associated with wearing or disposing of eyewear frames, please consult the Centers for Disease Control and Prevention at [www.cdc.gov](http://www.cdc.gov) for the most recent developments and advice. All information is provided based on the L'Amey America understanding and belief as of the date on which this information was prepared and is subject to change or modification at any time in L'Amey America sole discretion. L'Amey America. assumes no obligation or liability for the information contained herein.

## HOW TO SANITIZE YOUR SUNGLASSES AND OPTICAL FRAMES


### CLEAN AND SANITIZE ALL FRAMES AFTER EACH USE



Wash your hands for 20 seconds or use a hand sanitizer.




Apply rubber or vinyl gloves.

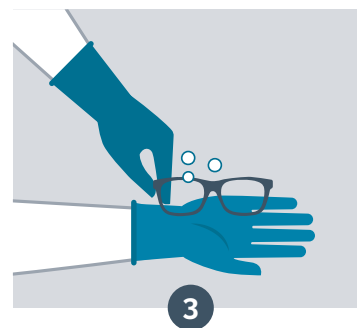

Clean every part of the frame/s with a hydrogen peroxide wipe or spray a hydrogen peroxide solution in a regular cleaning cloth.



Do not touch your mouth, nose or eyes during the process.

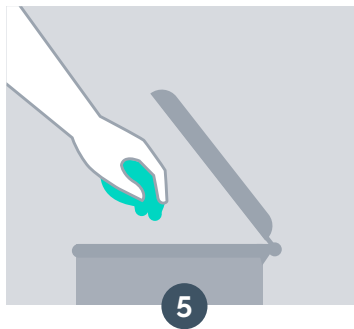

After cleaning the frames or interaction with patients, dispose the gloves and wipes. Do not reuse the cleaning cloth before machine washing it first.



Wash your hands for 20 seconds or use a hand sanitizer after disposing the gloves.

DISCLAIMER - The above information is provided for reference purposes only and does not constitute medical advice. If you have medical related concerns about the spreading and/or contracting of infectious diseases associated with wearing or disposing of eyewear frames, please consult the Centers for Disease Control and Prevention at [www.cdc.gov](http://www.cdc.gov) for the most recent developments and advice. All information is provided based on the L'Amey America understanding and belief as of the date on which this information was prepared and is subject to change or modification at any time in L'Amey America sole discretion. L'Amey America. assumes no obligation or liability for the information contained herein.